

TRUBYTE® PORTRAIT® IPN® SHADE CONVERSION CHART

For your convenience, the following chart provides a simple reference between several brands of anterior teeth and Vita® shades.

SHADE MATCH		SHADE CORRELATION			
Portrait IPN	Vita®	Bioform® IPN	Bioblend® IPN	Trublend® SLM®	Ivoclar® Chromascop
P1	A1	B51		T9	120
P2	A2	B53	104	T11	130
P3	A3	B54	108	T13	140
P3.5	A3.5	B83	116	T19	230
P4	A4	B84	118	T20	340
P11	B1	B59	100	T2	120
P12	B2	B52		T22	120
P13	B3	B55		T24	210
P14	B4	B56	110	T23	320
P21	C1	B91	102	T10	120
P22	C2	B94		T6	430
P23	C3	B95		T7	510
P24	C4	B96		T8	520
P32	D2	B92		T3	130
P33	D3	B93		T14	410
P34	D4	B69	113	T5	440
P59		B59	100	T2	
P62		B62	102	T10	
P65		B65	106	T15	
P66		B66	112	T12	
P67		B67	109	T16	
P69		B69	113	T5	
P77		B77		T17	
P81		B81	114	T18	
PW4*	*Portrait white shades, PW4 and PW7, correlate to illumine® i-4 and i-7.				
PW7*					

Portrait IPN teeth were tested with over one thousand dentists and laboratory technicians to assure that they match the Vita Lumin® Vacuum shade guide. Due to distinctive blend and shade characteristics of each brand of teeth, exact comparisons are not possible.

TRUBYTE, PORTRAIT, IPN, TRUBLEND, SLM, BIOFORM, BIOBLEND, ILLUMINÉ, the prefix "T" or "P" and a number, are trademarks of DENTSPLY International Inc. Vita, Ivoclar and Chromascop are not trademarks of DENTSPLY International Inc.

TRUBYTE® PORTRAIT® IPN® SHADE GUIDE AND DENTURE TEETH

The Portrait IPN plastic shade guide is an important aid in the denture teeth shade selection process, specifically designed for selection of Trubyte Portrait IPN artificial denture teeth. It offers complete shade availability cross-matched to the two most popular shade systems: Vita® A1-D4, and Trubyte Bioform® B59-B81.

The first 16 shades are designated "P" shades from 1 to 34, which cross match to Vita A1-D4 shades. Within these 16 shades are four hue groups: A = Reddish-Brown; B = Reddish-Yellow; C = Grayish-Shades; D = Reddish-Gray. Within each hue group are shades which have increasing hue (see reverse).

Portrait IPN teeth were researched extensively with over 1,000 dentists and laboratory technicians to assure the best possible cross match to the Vita Lumin® Vacuum shade guide.

The next 8 shades are designated "P" shades from P59 to P81, which correlate to the original Bioform "Base 8" shades. These shades present a new improved Portrait IPN blend rated aesthetically superior to Bioform IPN shades.

The last 2 shades, PW4 and PW7, are the two newest additions to the Portrait IPN family. These white shades were created to meet the demands of the bleaching market. PW4 and PW7 most closely tie into the Vita B and Vita A hue range respectively.

Portrait IPN plastic anterior teeth mimic natural dentition. That is, a natural variance of color from central to lateral to cuspid is carefully crafted into each set of teeth. Central incisors in each set are intended to closely match the Portrait IPN shade guide. Each tooth has a blend of 3 colors, from a contrasting neck to a translucent enamel incisal.

Shade matching is best done with a color corrected fluorescent light having a Color Rendering Index above 90 or in natural daylight. This type of light source was used in the research described above. For a variety of technical reasons, such as viewing and illuminating conditions, the perception of the shade match may vary.

The Trubyte Portrait IPN plastic shade guide is just that, a guide. However, it is the best representation of Trubyte Portrait IPN plastic anterior teeth available.

Premium Plastic Anteriors

Trubyte Portrait IPN Anteriors

Use it to correlate shades with:

Premium Plastic Posteriors

Trubyte Portrait IPN 40° Euroline®
Trubyte Portrait IPN 33°
Trubyte Portrait IPN 22° BioStabil®
Trubyte Portrait IPN 20°
Trubyte Portrait IPN 10° Anatoline®
Trubyte Portrait IPN 0°

Creating Natural Smiles®

DENTSPLY
TRUBYTE

DENTSPLY International Inc.
York, PA 17405-0872

Printed in U.S.A. 5/04 902787 Rev. J No. 4155-E
www.trubyte.dentsply.com or www.dentsply.com